# ENGLISH

## Intended Use:

A digital thermometer is a device used to measure the body temperature of a patient by means of a transducer coupled with an electric signal amplification, conditioning, and display unit.

## Safety Information:

- High, prolonged fever requires medical attention especially for young children. Please contact your physician.
- Do not allow children to take their temperatures unattended.
- Read the enclosed instructions carefully to ensure accurate temperature readings. Temperature readings are affected by many factors including exercise, drinking hot or cold beverages prior to taking a reading, as well as technique.
- Do not allow children to walk or run during temperature taking. The rigid probe could present a hazard. Oral, underarm, and rectal temperatures vary by approximately 1°F (0.5°C).
- **WARNING:** Keep battery out of children's reach. Swallowing battery could be harmful. Battery should not be recharged or placed into extreme heat as it may explode.
- **CAUTION:** Do not take an axillary temperature through patient's clothing. Direct probe cover to skin contact is required.
- **CAUTION:** Biting the probe tip while taking a temperature may result in damage to the probe.
- **CAUTION:** Do not autoclave. Please note the cleaning procedures described in this manual

## How to take temperature:

1. Place a probe cover carefully over the tip (Figure 1). Select oral, underarm or rectal as your measurement site based on the individual requiring a temperature measurement.
2. Push the Multi-Function button located beside the display window. The thermometer will beep and the display will read: "188.8 °F". This is a function check indicating the unit is working properly. Next the thermometer will read "ok" and then "Lo °F" (or "Lo °C").
3. The °F or °C symbol will flash when the unit is ready to measure temperature. Please note that the "—" indicates a low battery. See instructions for replacing battery before measuring temperature. Do not expect an accurate temperature reading if this low battery indicator is present. An "ERR" message in the window indicates your unit is not functioning properly.
4. Depending on the measurement site, the unit will complete measurement in 10 (rectal), 15 (oral), or approximately 20 (axillary) seconds. The °F or °C symbol will stop flashing and the unit will beep signaling completion.
5. Press the On-off button for approximately 1.5 seconds. The last temperature will be recalled. Record the temperature and time for reference. The reading will automatically be stored in the memory.
6. Push the Multi-Function button to turn the unit off. This will prolong the battery life. If you do not turn the unit off, it will shut off automatically in approximately 9 minutes.

Figure 1

## Oral use: (for children 4 years and older)

1. Do not drink hot or cold fluids, exercise or perform other activities that will raise or lower temperature readings when compared to your normal, average temperature. The mouth should remain closed up to 5 minutes before attempting a reading.
2. Place the probe tip well under the tongue as indicated (Figure 2).
3. Instruct children not to bite down on the probe.
4. The peak temperature should be reached in approximately 15 seconds. Opening the mouth or improper placement of the probe tip could result in a longer time for a reading.

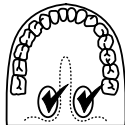
Figure 2

## Rectal use:

1. Commonly used for babies, young children or when it is difficult to take an oral or underarm temperature.
2. After placing the probe cover on the thermometer, lubricate with a water-soluble jelly or petroleum jelly.
3. Gently insert the probe tip no more than 3/4 inch or 2-3 cm into the rectum. Do not force the tip into the rectum if resistance is encountered.
4. Dispose of the used probe cover and wash the thermometer as recommended.
5. The peak temperature should be reached in approximately 10 seconds.


## Underarm (Axillary) use:

1. Wipe underarm with a dry towel.
2. Place probe tip in armpit and bring arm down against the body.
3. The probe should be pointing upward in the same direction as your body if possible.
4. The time required for an accurate underarm temperature may be 20 seconds or longer depending on placement of the probe and closure of the arm against the body.

**Changing Measurement Scale:** From the OFF position, press the Multi-Function button to activate the unit. Immediately after releasing the Multi-Function button, press the Multi-Function button again and hold for 2 seconds. The thermometer will automatically change from °F to °C or °C to °F. Release the Multi-Function button.

## Changing the battery:

When the "—" appears in the lower right of the display, the battery is exhausted and needs replacing. Make sure a battery of the same type is on hand and proceed as follows: Remove battery compartment lid. Use a small pointed object, but not sharp metal. Withdraw the battery. Replace with a new button battery (1.55v, LR41 or SR41) with the Cathode side (-) upward. Carefully replace the battery compartment lid securely. Properly dispose of the batteries, keeping them from small children and heat.

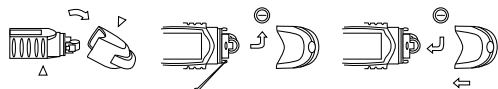## Standards:

Complies with EN 12470-3:2000 and ASTM E 1112-00

## How to care for this product:

- Use of probe covers is recommended to help avoid spreading germs, and to keep the thermometer clean. The thermometer should be cleaned after use even when probe covers are used.
- Clean the thermometer tip and probe only using isopropyl alcohol or soap and water.
- Store the unit in the protective case while not in use.
- Do not store or keep in direct sunlight, or in high temperatures.
- Keep battery away from children.
- Do not open the thermometer except to replace the battery.
- Do not drop the thermometer, as this may ruin the unit.
- The manufacturer recommends verifying the accuracy by an authorized laboratory every 2 years.

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
# Adtemp™ IV 415 Digital Fever Thermometer

## USE, CARE & MAINTENANCE

**ADC**  
AMERICAN DIAGNOSTIC CORPORATION

Specifications	
Size/Taille/Talla:	5.32"L x .86"W x .42"H (135mm L x 21.8mm W x 10.7mm H)
Weight/Poids/Peso:	0.493 oz (14 grams)
Display: Affichage: Visualización:	LCD, Liquid Crystal Display LCD, Affichage à cristaux liquides LCD, Indicador de cristal líquido
Battery/Batterie/Batería:	1.55v (LR41) Alkaline
Battery Life: La Vie de Batterie: Vida De la Batería:	Estimated 4,000 uses Estimé 4,000 utilisations Estimado 4,000 aplicaciones
Measurement Time: Temps de Mesure: Tiempo De la Medida:	Oral 15 Seconds / Rectal 10 Seconds / Axillary 20 Seconds 15 secondes orales / 10 secondes rectales / 20 Secondes Axillaires 15 segundos orales / 10 segundos rectales / 20 Segundos Axilares
Measurement Range: Chaîne de Mesure: Rango De la Medida:	90.0°F to 111.9°F / 32.0°C to 43.9°C (±0.2°F/±0.1°C) in full measuring range
Measurement Mode: Mode de Mesure : Modo De la Medida:	Real measurement with peak-hold method La vraie mesure avec crête-tienement la méthode La medida verdadera con pico-sostiene método
Audible Alarm: Alarme Sonore: Alarma Audible:	1 beep at start-up. Continuous beep at measurement completion 1 bip à la mise en train. Bip continu à l'accomplissement de mesure 1 señal acústica en el start-up. Señal acústica continua en la terminación de la medida
Auto Turn Off: Automatique Éteignez: Auto Dé vuelta Apagado:	In approximately 9 minutes En approximativement 9 minutes En aproximadamente 9 minutos
Memory Feature: Dispositif de Mémoire: Característica De la Memoria:	Stores last temperature taken. Les mémoires durent la température prise. Los almacenes duran la temperatura tomada.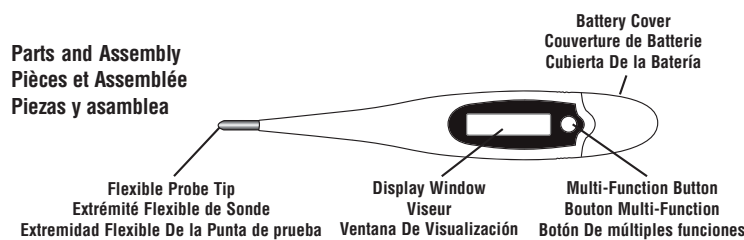

**WARRANTY:** This Thermometer is guaranteed for a period of 24 months from date of purchase. This warranty does not cover batteries, or damage resulting from improper handling or use. Specific legal rights stipulated in your state may guide this warranty. If you have a legitimate claim under this warranty, send item(s) postage paid to Service Center, Attn: Repair Dept., 55 Commerce Dr., Hauppauge, NY 11788. Please include your name and address, phone no., proof of purchase, a brief note explaining the problem., your return address and \$2.00 to cover shipping and handling.

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**ADC**  
AMERICAN DIAGNOSTIC CORPORATION

ADC  
55 Commerce Drive  
Hauppauge, NY 11788

ADC (UK) Ltd.  
Unit 6, PO14 1TH  
United Kingdom

IB p/n 93-415-00 rev 1

0197

Inspected in the U.S.A.  
Made in China  
tel: 631-273-9600, 1-800-232-2670  
fax: 631-273-9659

www.adctoday.com  
email: info@adctoday.com

Printed in China