


# Assembly Instructions for PROSCOPE™ Teaching Stethoscopes

(Models #671 and #661)

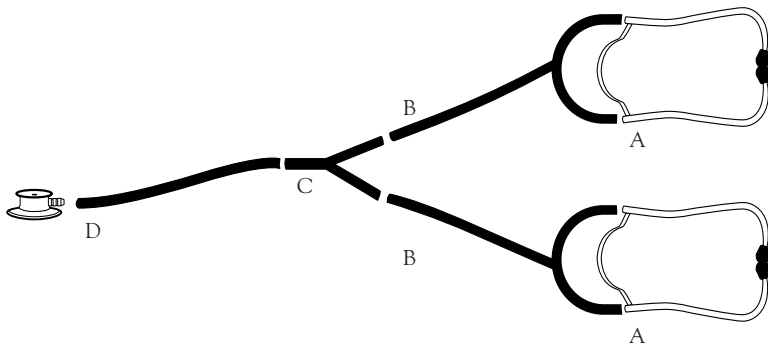

## TO ASSEMBLE

- A. Attach "Y" tubing (#660-05BK) to each set of the binaurals (660-05) as shown in the illustration.
- B. Attach free end of each "Y" tubing to the double pair end of the "Y" connector (#611-20).
- C. Connect one end of the straight tubing (#671-05TBK) to single end of the "Y" connector.
- D. Connect remaining end of the straight tubing to stem of chestpiece (#660-01 or #670-01).

Make sure all tubing connections are firm.

**PROSCOPE™ Teaching Stethoscopes**  
**(Models #671 and #661)**  
**Replacement Parts Included**

| <b>Part Number</b> | <b>Description</b>               | <b>Qty.</b> |
|--------------------|----------------------------------|-------------|
| 660-01             | PROSCOPE™ Nurse Chestpiece       | 1*          |
| 670-01             | PROSCOPE™ Dual Head Chestpiece   | 1*          |
| 611-20             | “Y” Connector                    | 1           |
| 671-05TBK          | PVC Tubing, Straight, 10”, Black | 1           |
| 660-05TBK          | PVC “Y” tubing, 22”, Black       | 2           |
| 660-05             | Binaurals                        | 2           |

**\*NOTE:** PROSCOPE™ Model #671 includes the 670-01 chestpiece only  
PROSCOPE™ Model #661 includes the 660-01 chestpiece only



55 Commerce Drive  
Hauppauge, NY 11788  
1-800-ADC-2670 • [www.adctoday.com](http://www.adctoday.com)



Attention, see instructions for use.

**CE** American Diagnostic Corporation, Ltd.,  
Hants, UK PO14 1TH