



Assembly


Box Contents/Parts

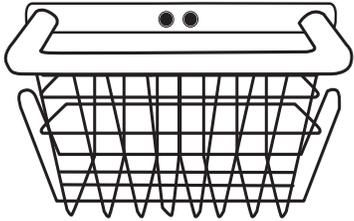

Basket
9005MBAS-00

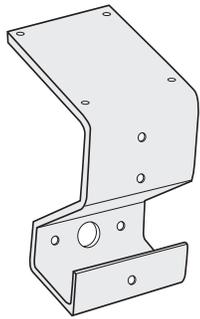


Wall Bracket
9005W-00

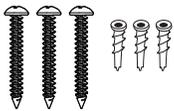

3 Wall Screws /
3 Mollys
5600-HW-01




2 Basket Bolts
9005MBAS-S-00

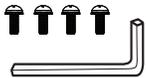

Hex Key
9005M-HW-00

1.

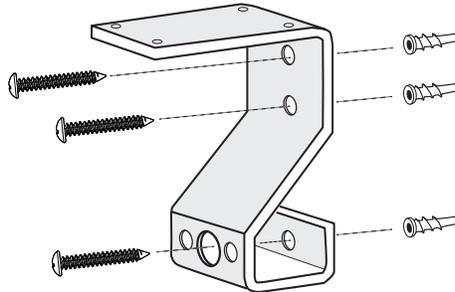


1.

Attach wall bracket to wall before assembling basket. For optimum viewing, it is suggested that you measure 60" from the floor to the top of the wall mounting bracket (or adjust as needed). Once positioned to desired height, mark screw positions on the wall making sure bracket is level. Screw in the three supplied screws (5600-HW-01) firmly into a stud, or use the three supplied mollys and secure to the wall.


2.

After the wall bracket has been attached to the wall, position the basket onto the bracket and screw the two basket bolts (9005MBAS-S-00) through the basket openings and tighten.

2.

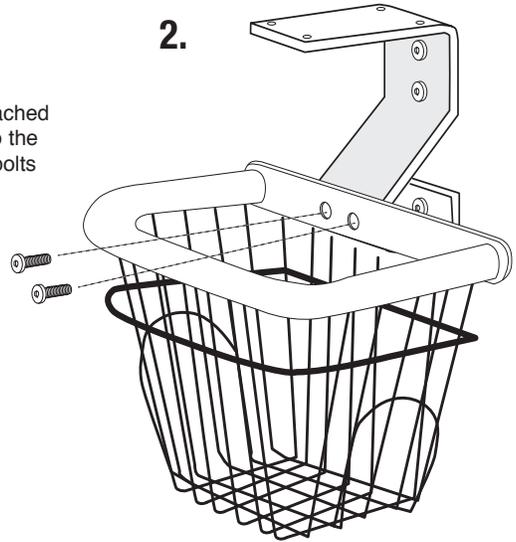

3.

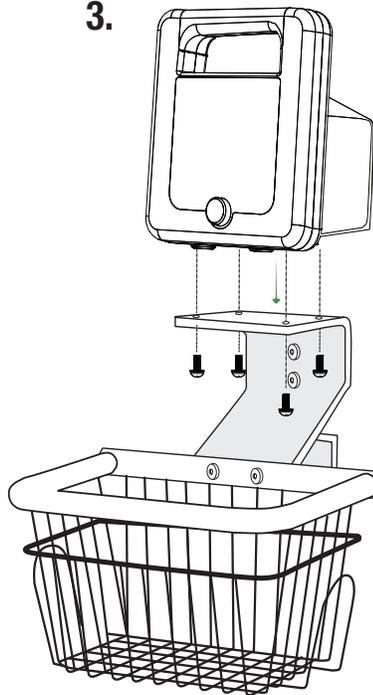

3.

Align holes on bottom of the ADView 2 with the four holes on the mounting bracket as shown. Insert the four hex mounting bolts (9005M-HW-00) through the bottom of mounting bracket into the ADView 2 and tighten.



American Diagnostic Corporation
55 Commerce Drive
Hauppauge, NY 11788

tel: 631-273-9600
toll free: 1-800-232-2670
fax: 631-273-9659

www.adctoday.com/adview2
info@adctoday.com

To register your product, visit us at
www.adctoday.com
and follow the links

For questions, comments, or suggestions,
call in the U.S.A. Toll Free:
1-800-ADC-2670