



1a. Attaching the Temp Probe Connector

With the black side facing down, slide the black connector at the end of the probe cord into the notches on the probe holder as shown.

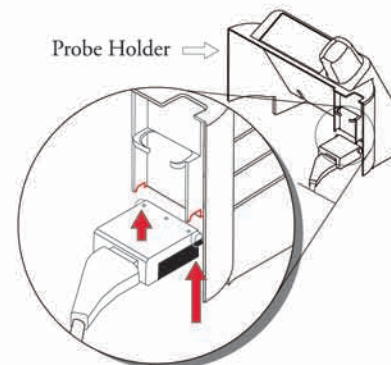


1b. Attaching the Temp Probe Connector

Rotate the connector up until it snaps into the retention notches (shown in red). The connector should fit snugly.

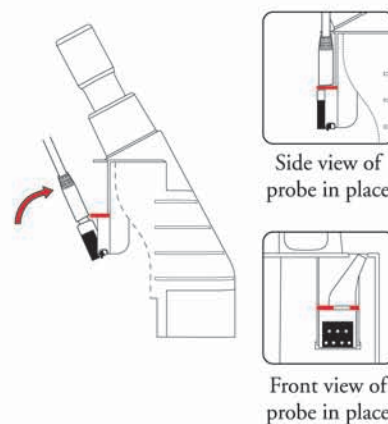

2. Assembling the Temp Module for Use

- Slide the probe cover box holder onto the temperature module.
- Place probe in well.
- Place the probe cover box into the holder.

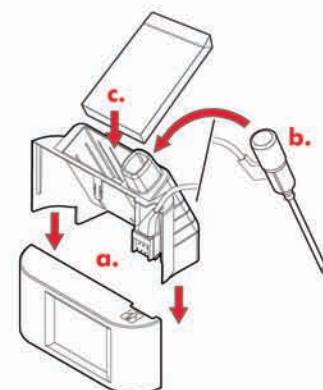

ADC
55 Commerce Drive
Hauppauge, NY 11788

Suntech Medical, Ltd.
Oakfield Industrial Estate
Eynsham, Oxfordshire
OX29 4TS UK

Inspected in the U.S.A.
Made in China
tel: 631-273-9600
toll free: 1-800-232-2670
fax: 631-273-9659

www.adctoday.com/adview
info@adctoday.com

To Register Your Product, visit us at
www.adctoday.com
and follow the links

For questions, comments, or suggestions,
call in the U.S.A. Toll Free:
1-800-ADC-2670