
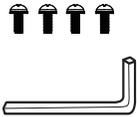


Assembly



Box Contents/Parts

Desktop Storage Caddy
9005D-00

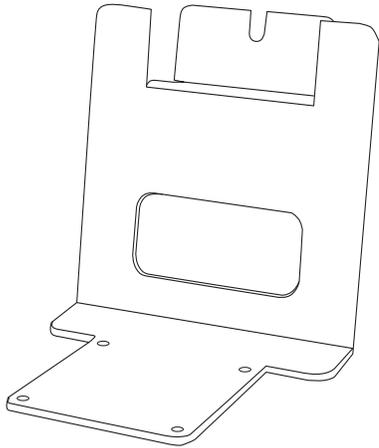

Hex Key/
 4 Screws
9005M-HW-00

1. Align ADView 2 on top of the Desk Caddy, so the four holes in the caddy line up with the screw holes on the bottom of the ADView 2 when seated.

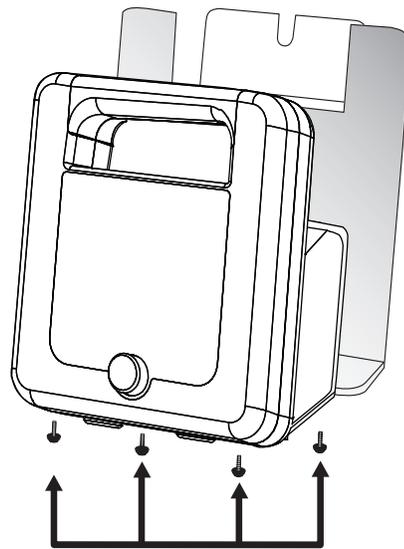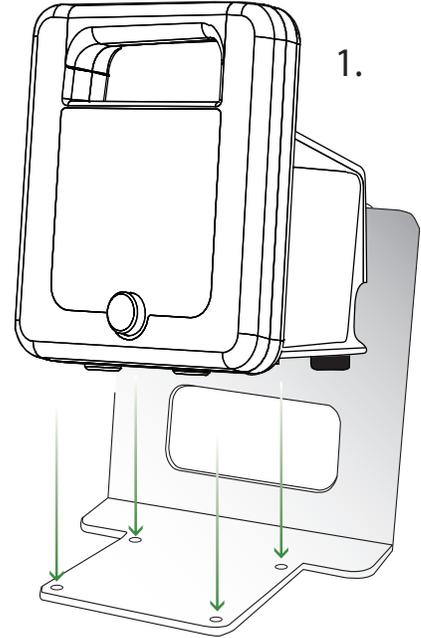

2. Once fully seated, use the hex key to screw in the four mounting screws through the bottom of the caddy into the ADView 2. Hand-tighten all four screws until firmly set.

When complete, you may begin to connect cabling and cuff tubing.



American Diagnostic Corporation
 55 Commerce Drive
 Hauppauge, NY 11788

tel: 631-273-9600
 toll free: 1-800-232-2670
 fax: 631-273-9659

www.adctoday.com/adview2
info@adctoday.com

To register your product, visit us at
www.adctoday.com
 and follow the links

For questions, comments, or suggestions,
 call in the U.S.A. Toll Free:
1-800-ADC-2670