



Hook & Loop Closure **TOURNIQUET** User's Manual

Intended Use

Tourniquets are non-inflatable (non-pneumatic) devices intended to be used as an effective way to achieve controlled vessel constriction. Latex-free tourniquet with Hook & Loop contact closure can be applied easily and instantly for perfect venous or arterial constriction.

Warnings

- Caution:** Serious injury or death may result from the improper use of this equipment.
- Caution:** This equipment has been designed and manufactured for use by experienced professionals only.
- Caution:** Do not attempt to use this equipment without proper training.
- Caution:** This product is only tested to withstand 30 uses.
- Caution:** This product cannot be sterilized and must be discarded if it becomes contaminated.

Instructions

1. When applying tourniquet, be sure label is face down on skin.
 2. Hold label firmly with one hand and at the same time, stretch the tourniquet to encircle the limb.
 3. Adjust to tension desired for clinical or first aid use, lock into place with Hook & Loop contact closure.
-

Care and Maintenance

Clean tourniquet with an alcohol wipe after each use.



ADC
55 Commerce Drive
Hauppauge, NY 11788



ADC (UK) Ltd.
Unit 6, PO14 1TH
United Kingdom

Inspected in the U.S.A.
Made in China
tel: 631-273-9600, 1-800-232-2670
fax: 631-273-9659

www.adctoday.com
email: info@adctoday.com

