

Adlite™ Pro

Instructions for Use

Thank you for choosing an ADC® ADLITE PRO™, a precision crafted LED penlight. The ADLITE PRO™ body is machined from brass then electroplated for a durable finish. The long lasting LED lamp is guaranteed for 10,000 hours of use.

Device Description and Intended Use:

This device is intended for general, non contact illumination.

General Warnings:

This instrument is NOT to be used in the presence of flammable anesthetics.

Operations:

To turn on, use thumb to slide the textured blue power switch away from the body of the penlight. To turn off, slide switch back towards the tip.

Battery Replacement:

ADLITE PRO™ uses 2 “AAA” alkaline batteries. To replace, twist LED lamp tip counter-clockwise. Replace both batteries from barrel, being careful to observe correct polarity, by inserting (-) negative ends in first. Reassemble lamp tip by twisting clockwise until tight.



Limited Warranty

American Diagnostic Corporation (ADC®) warrants its products against defects in materials and workmanship under normal use and service as follows:

1. LED lamp is warranted for life.
2. ADLITE PRO™ Penlight is warranted for 2 years.
3. Warranty service extends to the original retail purchaser only and commences with the date of delivery.

What is Covered: Replacement of parts, and labor.

What is Not Covered: Transportation charges to and from ADC®. Damages caused by abuse, misuse, accident, or negligence. Incidental, special, or consequential damages. Some states do not allow the exclusion or limitation of incidental, special, or consequential damages, so this limitation may not apply to you.

To Obtain Warranty Service: Send item(s) postage paid to ADC®, Attn: Repair Dept., 55 Commerce Dr., Hauppauge, NY 11788.

Please include your name and address, phone no., proof of purchase, and a brief note explaining the problem.

Implied Warranty: Any implied warranty shall be limited in duration to the terms of this warranty and in no case beyond the original selling price (except where prohibited by law). This warranty gives you specific legal rights and you may have other rights which vary from state to state.

This manual is available online in a variety of languages,
follow the links for language options:

www.adctoday.com/care

FOR QUESTIONS, COMMENTS, OR SUGGESTIONS
CALL TOLL FREE:

1-800-ADC-2670 OR VISIT:
www.adctoday.com/feedback



ADC
55 Commerce Drive
Hauppauge, NY 11788



ADC (UK) Ltd.
Unit 6, PO14 1TH
United Kingdom

Inspected in the U.S.A.

Made in China

tel: 631-273-9600

toll free: 1-800-232-2670

fax: 631-273-9659

www.adctoday.com

info@adctoday.com

