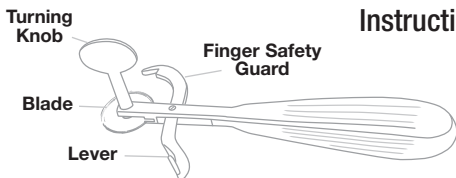

ADC[®]

380N Ring Cutter

Instructions for Use



How to Use

- Step 1:** Cleanse the finger with a mild soap or antiseptic to prevent infection.
- Step 2:** Slide the finger guard under the ring, between the ring and the finger.
- Step 3:** Position the blade over the ring, making sure the finger is protected.
- Step 4:** Squeeze the lever to apply slight pressure from the blade onto the ring.
- Step 5:** Twist the turning knob clockwise, rotating the blade until it cuts through the ring and releases it.
- Step 6:** Slide guard out from finger and ring.



If the tension is too great to spread the ring after first cut, another cut 180° from the original cut site can be made. This second cut allows the ring to fall off in two pieces.

Look for signs of finger ischemia, such as mottling, blue-gray or white color of the distal digit, or severe pain, as this indicates an urgent need to remove the ring.

Replacing the blade

Wear gloves for safety, especially if the blade is damaged or sharp.

- Step 1:** Hold blade with tweezers or pliers and twist the turning knob counterclockwise to loosen the blade.
- Step 2:** Carefully lift blade out of its slot. Dispose of the old blade properly, as it may still have sharp edges.
- Step 3:** Align replacement blade (pn #381N) carefully in its slot, ensuring cutting edge is facing outward. While holding the blade in place, twist the turning knob clockwise to secure it tightly.

Step 4: Perform a test cut on a safe material to confirm the replacement blade is securely aligned and cutting smoothly.

Cleaning and Disinfecting

Disassemble the blade and handle carefully.

Rinse the parts thoroughly under warm water to remove any debris or contaminants.

Prepare a solution of mild soap and water. Using a soft brush or cloth gently scrub the cutter, blade and joints where debris can accumulate.

Wipe down the ring cutter with a disinfectant solution, such as 70% isopropyl alcohol or another medical grade disinfectant.

Dry all parts of the ring cutter with a clean, lint free cloth to prevent rust or corrosion. Store in a dry, safe location to avoid damage.

Limited Warranty

ADC warrants its products against defects in materials and workmanship under normal use and service as follows:

1. Warranty service extends to the original retail purchaser only and commences with the date of delivery.
2. Your ADC ring cutter is warranted for one year from date of purchase (all parts).

What Is Covered: Replacement of parts, and labor.

What Is Not Covered: Transportation charges to ADC. Damages caused by abuse, misuse, accident, or negligence. Incidental, special, or consequential damages. Some states do not allow the exclusion or limitation of incidental, special, or consequential damages, so this limitation may not apply to you.

To Obtain Warranty Service: Send item(s) postage paid to ADC, Attn: Repair Dept., 55 Commerce Dr., Hauppauge, NY 11788. Please include your name and address, phone no., proof of purchase, and a brief note explaining the problem.

Implied Warranty: Any implied warranty shall be limited in duration to the terms of this warranty and in no case beyond the original selling price (except where prohibited by law). This warranty gives you specific legal rights and you may have other rights which vary from state to state.

**To register your product visit us at
www.adctoday.com/support/warranty-registration**



 ADC®
55 Commerce Drive
Hauppauge, NY 11788
U.S.A.

Inspected, and packaged in the U.S.A.

Made in Pakistan

tel: 631-273-9600
toll free: 1-800-232-2670
fax: 631-273-9659

www.adctoday.com
info@adctoday.com

