

# Diagnostix™

## PMV Otoscope

### Specula, Focusing and Cleaning Instructions



#### **Attaching and removing ear specula:**

To equip the otoscope head, single-use ear speculum, turn the speculum clockwise until you can feel a resistance.

To remove the speculum, twist off the speculum by turning it counterclockwise.

#### **Focusing wheel:**

The focusing wheel enables you to adjust the focusing range in the auditory canal (tympanic membrane).

The adjustment wheel moves up and down to focus the lens.

# Diagnostix™

## PMV Otoscope



### Cleaning and disinfection

The instrument head with handle can be cleaned externally with a moist cloth until visually clean. Wipe disinfection as specified by the disinfectant manufacturer. Only disinfectants with proven efficacy should be used, taking into account the national requirements. After disinfection, wipe the instrument down with a moist cloth to remove possible disinfectant residues. The components that come into contact with the skin can be rubbed down with alcohol or a suitable disinfectant.

### CAUTION!

- Never immerse the instrument head with handle in any liquid! Take care to ensure that no liquids get inside the casing!
- This item is not approved for automated reprocessing and sterilization. These procedures cause irreparable damage!