

# Diagnostix™

## ADSTATION

Modular Diagnostic System™

---

### Specula Extension Assembly Instructions

After taping the specula drilling template in the desired position on the wall, drill the 2 appropriate holes (7/32" (5.556 mm) diameter, and 1.25" (32 mm) deep as indicated. Push the supplied mollies as far as they will go into the drill holes in the wall.

Put the spacer sleeves supplied (1) onto the dome (2) of the holder (3).

The screws should now be pushed into the holes provided (4) in the holder (3).





Place the holder on the wall so that the screws line up with the plugs and screw them in tightly.

When the holder is firmly in place on the wall, the casing should be attached to the holder as follows.

Place the casing on the holder such that the T-grooves (5) lie exactly over the T-groove guides (6), the casing should now be pushed evenly backwards until it snaps into position. The casing should be level with the viewing windows of the holder.

Place the lid (7) in the intended location (8).

The specula can now be filled as desired.

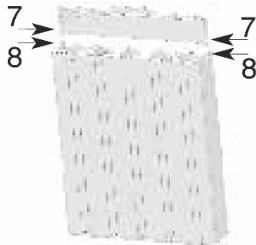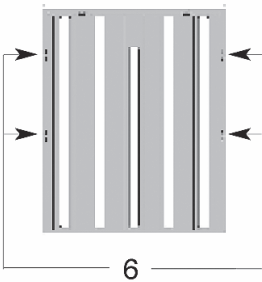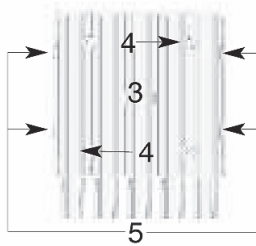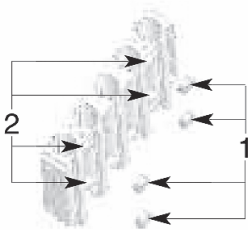

ADC  
55 Commerce Drive  
Hauppauge, NY 11788



ADC (UK) Ltd.  
Unit 6, PO14 1TH  
United Kingdom



Consult instructions  
for use



Type BF applied part



IB p/n 93-5630-00

Made in Germany  
Inspected and Packaged in the U.S.A.

tel: 631-273-9600  
toll free: 1-800-232-2670  
fax: 631-273-9659

[www.adctoday.com](http://www.adctoday.com)

email: [info@adctoday.com](mailto:info@adctoday.com)



Printed in U.S.A.