

# Diagnostix™ ADSTATION™

Modular Diagnostic System

---

## Specula Extension Assembly Instructions

1. Position the specula drilling template on the wall where desired and tape into place.
2. With a pencil, mark two holes on the wall where indicated, then remove template from the wall.
3. Using a 7/32" bit, drill holes where marked, about 1 1/4" deep. Push the supplied screw anchors into the holes until flush against the wall.
4. Take the dispenser portion of the unit (Figure 1) and check to ensure the spacer caps are in place on the back. Push the two supplied screws into the holes on the front of the unit and position it in on the wall in line with the anchors. Screw snugly into place.
5. Next attach the faceplate (Figure 2). To do so, align the four grooved tracks on the sides of the faceplate with the corresponding guides on the dispenser. Push firmly until the faceplate clicks into place.
6. To attach the lid (Figure 3), align the lid tabs with the corresponding holes on the faceplate until the lid rests in position.
7. Fill the dispenser as needed.

**See illustrations on reverse side.**

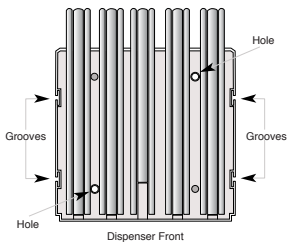


Figure 1

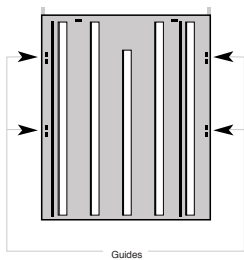
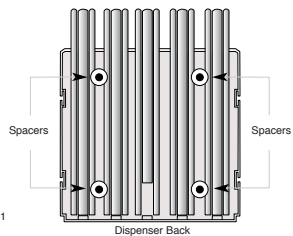


Figure 2

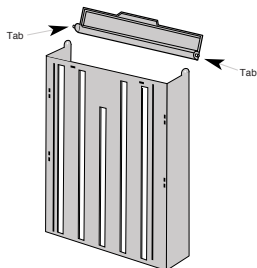


Figure 3



American Diagnostic Corporation  
 55 Commerce Drive  
 Hauppauge, NY 11788 USA  
 1-800-ADC-2670  
[www.adctoday.com](http://www.adctoday.com)

tel: 631-273-9600  
 toll free: 1-800-232-2670  
 fax: 631-273-9659

[www.adctoday.com](http://www.adctoday.com)  
 email: [info@adctoday.com](mailto:info@adctoday.com)