


Special Installation Notes

TRANSFORMER COMPATIBILITY

Transformer (Back Side)

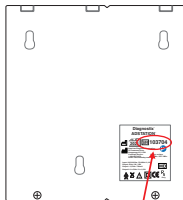


SERIAL NUMBER

If you have purchased an extension unit to use with an **older Adstation transformer**, please check the transformer serial number before installation. The extension unit in this shipment is compatible only with a transformer whose serial number is 100000 or higher. Damage will occur if paired with an older transformer.

If this purchase includes both a transformer and an extension module, we have already confirmed compatibility.

ELECTRIC CONNECTION CABLE

After connecting modules with the electric cable (see Wallboard Installation Instructions), ensure excess cable is tucked securely within the module frame before mounting on wall or wallboard.

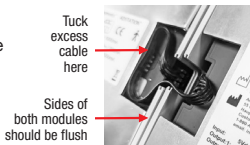

Warranty: Diagnostix Adstation transformers and extension modules are warranted for five years from date of purchase. The warranty does not apply to damage caused by improper handling, accidents, improper use, or alterations made to the instrument by third parties. Damage caused by connection of incompatible extensions modules is not covered; please confirm compatibility of all components before installing. The warranty is only valid after the product is registered online at: www.adctoday.com/support/warranty-registration.



American Diagnostic Corporation
 55 Commerce Drive, Hauppauge, NY 11788

tel: 631-273-9600
 toll free: 1-800-232-2670
 fax: 631-273-9659

www.adctoday.com
 email: info@adctoday.com

To register your product, visit us at
www.adctoday.com
 and follow the links

For questions, comments, or suggestions,
 call in the U.S.A. Toll Free:
1-800-ADC-2670