


Stethoscope ID Tag


Personalized ID Tag

ADC®'s proprietary ID tag fits both Adscope® and Proscope™ models. Made of two piece injection molded plastic, that snaps tightly together around tubing.

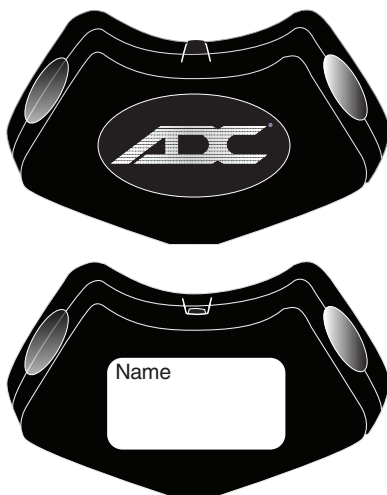

55 Commerce Drive • Hauppauge, N.Y. 11788
www.adctoday.com

Proscopes™ - Center the “Y” tubing in the 3 openings of the rear half, (2 sides and bottom). After positioning as shown, snap front and back halves securely together until the tab locks into place enclosing tubing completely. (Figure 1)

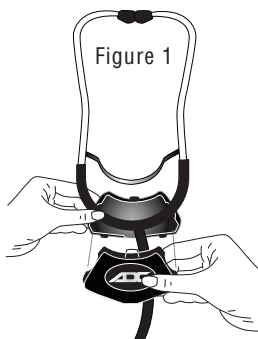

Adscopes® - For Adscopes® that feature Bi-lumen tubing, (Models 600, 601, 602, 606, 658) position ID tag as shown in Figure 2. For all other Adscope® models, position ID tag as shown in Figure 3. After positioning as shown, snap front and back halves securely together until the tabs lock in place - enclosing tubing completely.

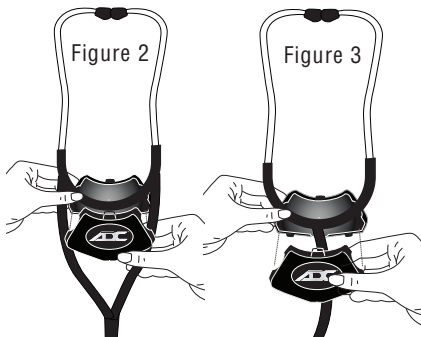


Figure 4

Optional - You can permanently add your name or initials on the rear panel (Figure 4).

