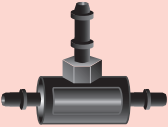


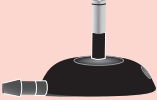


CUFF CONNECTIVITY

Congratulations on the purchase of your new ADC® replacement cuff for digital blood pressure home care units. You may have received 2 connectors. If so, use the diagram below to choose the correct connector to be used with your specific BP monitor. When connecting tubes to the appropriate connector, use a slight twisting motion and make sure tube and connector are firmly seated. Use the same slight twisting motion when inserting connector into monitor.

IMPORTANT: Using the wrong connector for your specific monitor may lead to measurement error. Refer to your Blood Pressure unit's Instruction Booklet for cuff connection specific to your model.

MONITOR MODEL	CONNECTOR	CONNECTORS	
6012, 6012N	897CN-1		
6013	897CN-2		
6014	897CN-2		
6017	897CN-2		
6021, 6021N	897CN-2		
6022, 6022N	897CN-2		
6023, 6023N	897CN-2		
6021NSA	897CN-5		
6021NSA rev 1	897CN-5N		