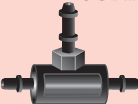



CUFF CONNECTIVITY

Congratulations on the purchase of your new ADC® replacement cuff for digital blood pressure home care units. If you received two connectors, use the chart below to choose the one that's right for your monitor. Using the wrong connector may lead to measurement error, so it's important to select the correct one. When inserting the connector into the tube, use a slight twisting motion and make sure both are firmly seated. Use the same slight twisting motion when inserting the connector into the monitor.

MONITOR MODEL	CONNECTOR	CONNECTORS	
6012, 6012N	897CN-1		
6021, 6021N	897CN-2		
6022, 6022N	897CN-2		
6023, 6023N	897CN-2		
6021NSA	897CN-5 897CN-5N2	