

REPLACEMENT CUFF FOR ADC ADVANTAGE HOME BLOOD PRESSURE MONITORS

This cuff is designed for use with newer ADC home blood pressure monitors:

- *Advantage 6112*
- *Advantage Plus 6121*
- *Advantage Connect 6024N*

This cuff can also be used with these legacy ADC home blood pressure models:

- *Advantage Semi-Automatic 6012N*
- *Advantage 6021N*
- *Advantage Plus 6022N*

To use the cuff with a legacy model, however, **you must first replace the connector on the end of the cuff tubing.** A replacement connector is included here.

To change the connector:

1. Remove the existing connector by pulling it away from the tubing while gently twisting.
2. Insert the new connector into the tubing and push while gently twisting until it is firmly seated.

To connect the cuff, simply insert the connector into the hole in the left side of the monitor.

Make sure the cuff connector is fully inserted before taking a reading.

If you have any questions or need to purchase additional connectors, please contact ADC Customer Service at 1-800-ADC-2670 or email info@adctoday.com.