

For advantage connect® 6024N/6025N Customers

If you will be using the enclosed cuff with an Advantage Connect 6024N or 6025N monitor, you must first replace the connector on the end of the cuff tubing. A compatible replacement connector is included in this box.

Remove the existing connector by pulling it away from the tube while gently twisting. To insert the new connector, push and twist until it is firmly seated in the tubing.

Using the wrong connector may lead to measurement error, so it's important to use the style designed for your monitor. If you have any questions, please contact ADC Customer Service at **1-800-232-2670** or info@adctoday.com



897CN-6