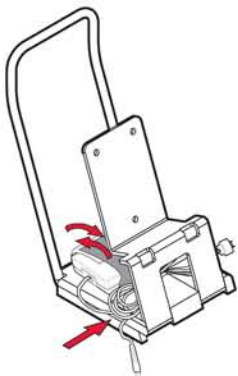
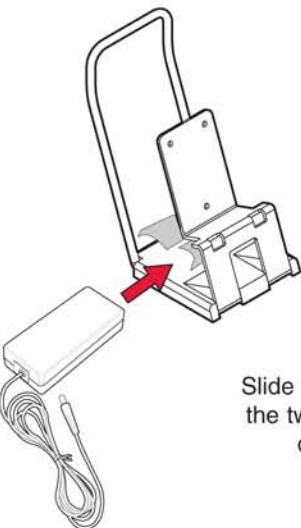



DESKTOP CADDY SETUP GUIDE


1.  Place Tabletop Stand on a Flat Surface.

4.  Store the power supply cord by sliding into the space between power supply and the front panel of the caddy (as shown).

2.  Slide Power Supply between the two Velcro® tabs on rear deck of the caddy.

5.  Place Adview on to caddy, carefully aligning the 3 holes on the unit with the 3 holes on the caddy. Using the thumb screws, secure the rear panel of the Adview to the caddy.

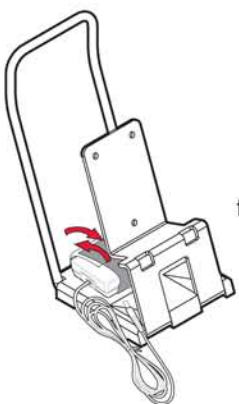3.  With Power Supply in place, fold over the Velcro® tabs and tightly secure the Power Supply in place.

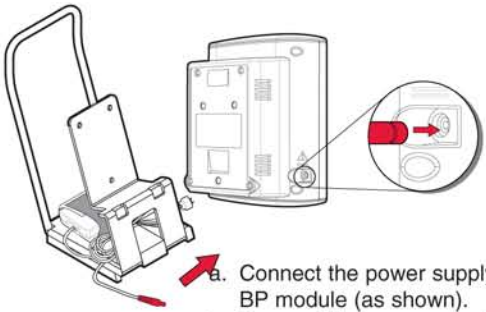6.  a. Connect the power supply to the BP module (as shown).
b. Connect the power supply to an AC main power source. The device will turn on automatically.

Before using the device, fully charge the battery for eight hours.



ADC
55 Commerce Drive
Hauppauge, NY 11788



Suntech Medical, Ltd.
Oakfield Industrial Estate
Eynsham, Oxfordshire
OX29 4TS UK

Inspected in the U.S.A.
Made in China
tel: 631-273-9600
toll free: 1-800-232-2670
fax: 631-273-9659

www.adctoday.com/adview
info@adctoday.com

To Register Your Product, visit us at
www.adctoday.com
and follow the links

For questions, comments, or suggestions,
call in the U.S.A. Toll Free:
1-800-ADC-2670