


The following instructions guide you in adding the pulse oximetry module to the **ADC AdView™ Modular Diagnostic Station**. Included in the Pulse Oximetry Module kit are the pulse oximetry module, the adult reusable sensor, and a 6' extension cable. The pulse oximetry module attaches to the bottom of the BP module.

Adding the Pulse Oximeter Module

1. Using the power button on the right side of the BP module, ensure that the **ADC AdView™ Modular Diagnostic Station** is off.
2. Remove the cover plate from the bottom of the main BP module.
3. Slide the pulse oximetry module along the guides on the bottom of the main BP module from front to back until it snaps into place. All the segments of the pulse oximetry module display will light when the modules have been connected correctly.



4. Turn the device on using the power button on the right side of the BP module. At the end of the start-up sequence, the pulse oximetry module display will be blank except for the "%SpO₂" symbol.
5. Turn the device off, connect the adult reusable sensor to the 6' extension cable, and then the cable to the connector on the module. Secure the cable to your device using the retention clip.

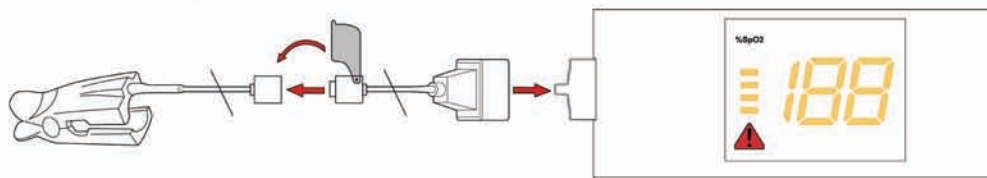

6. Turn the device on. All display segments light up for three to five seconds. A short beep indicates that the **ADC AdView™ Modular Diagnostic Station** is ready.

Note: If the pulse oximetry module does not appear to be working properly, cycle the power several times using the power button on the right side of the BP module. This will “synchronize” all the modules. The modules are synchronized when, after you turn the power on, all segments on all modules display light simultaneously for 3 to 5 seconds, followed by a short beep, and all displays go to their “ready” state (BP: battery icon and middle segments of the systolic value are lit; Temperature: appropriate temperature units icon is lit; Pulse Oximetry: the “%SpO2” icon is lit).

Refer to the **ADC AdView™ Modular Diagnostic Station** user’s manual for operation, maintenance and safety instructions.



ADC
55 Commerce Drive
Hauppauge, NY 11788



Suntech Medical, Ltd.
Oakfield Industrial Estate
Eynsham, Oxfordshire
OX29 4TS UK

Inspected in the U.S.A.
Made in China
tel: 631-273-9600
toll free: 1-800-232-2670
fax: 631-273-9659

www.adctoday.com/adview
info@adctoday.com

Printed in the USA

To Register Your Product, visit us at
www.adctoday.com
and follow the links

For questions, comments, or suggestions,
call in the U.S.A. Toll Free:
1-800-ADC-2670